## 3M ESPE

## Adper Prompt L-Pop

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- Notes** It is extremely important for adhesion that both adhesive components be completely mixed. Using incompletely activated Adper Prompt L-Pop blisters can cause the adhesive effect to be eliminated. Massaging the adhesive only partially or shortening the massage time also compromises adhesion.

**Disposal** Only dispose of completely empty packages. The residue of the etching fluids – if heavily diluted with water or neutralized – can be poured down the drain.

**Incompatibilities** In sensitive individuals, sensitization to the product cannot be excluded. If allergic reactions occur, use of the product should be discontinued and the product completely removed. In rare cases, particularly when the instructions for use are not followed, pulp discoloration is possible.

**Storage and Shelf Life** Adper Prompt L-Pop blisters should be protected from light. Do not store the product at temperatures above 25°C. Do not use after the expiration date.

**Customer Information** No person is authorized to provide any information which deviates from the information provided in this instruction sheet.

**Warranty** 3M ESPE warrants this product will be free from defects in material and manufacture. 3M ESPE MAKES NO OTHER WARRANTIES INCLUDING ANY IMPLIED WARRANTY OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE. User is responsible for determining the suitability of the product for intended application. If this product is defective within the warranty period, your exclusive remedy and 3M ESPE’s sole obligation shall be repair or replacement of the 3M ESPE product.

**Limitation of Liability** 3M ESPE will not be liable for any loss of profit or consequential damages, whether direct, indirect, special, incidental or consequential, regardless of the theory asserted, including warranty.

**Last update:** 02/03

**ENGLISH**

**Product Description** Adper Prompt L-Pop, manufactured by 3M ESPE, is a self-etch adhesive in L-Pop blisters for one-time use. Both components in the blister packing are activated, making the solution ready for use. Etching, priming, and bonding take place in one step while it is being brushed on. The adhesive is polymerized with halogen, laser, LED, or plasma lights.

⚠ Instructions for Use should not be discarded for the duration of product use.

### Indications

- Bonding between dentin/enamel and composite filling materials
- Bonding between dentin/enamel and composite filling materials
- Bonding of light-cure pit and fissure sealants

### Contraindications

Because of the high bond to enamel and the attendant risk of enamel damage upon bracket removal, the use of Adper Prompt L-Pop for orthodontic applications is contraindicated.

Adper Prompt L-Pop is suited for purely light-curing materials, but not for dual-curing or self-curing materials.

Eugenol-free products should be used, as retining materials that contain eugenol, can adversely affect the polymerization of the adhesive.

### Precautions

- Avoid contact with mucosa, skin, and eyes. In the event of accidental contact, the affected area should be thoroughly rinsed with water.
- Wear gloves and eye protector during processing.
- Use a rubber dam if necessary. Superficial changes of the gingiva, which may occur in some cases, are reversible.
- Eugenol-free products should be used, as retining materials that contain eugenol, can adversely affect the polymerization of the adhesive.

### Preparation

- Remove loose preparation debris by spraying with water.
- Use a 2-3 second burst of air to dry the cavity. The cavity should only be dried to the extent that there is no water visible on the surface. Over-drying may result in lost operative sensitivity.

- Clean untreated tooth structure with a rubber cup and an oil and fluoride cleaning paste (e.g., pumice paste), rinse thoroughly with water spray and dry.

### Pulp Protection

To prevent pulp irritation, areas of deep dentin should be covered with calcium hydroxide preparation (e.g., Alkaliner<sup>®</sup>, manufactured by 3M ESPE). The calcium hydroxide preparation should be covered with a retining material, e.g. Vitrebond<sup>®</sup>, manufactured by 3M ESPE.

### Mixing Adper Prompt L-Pop

Attention: Do not activate Adper Prompt L-Pop without a disposable applicator.

- Hold the shaft of the disposable applicator with one hand.
- With the thumb and index finger of the other hand, squeeze the red fluid reservoir, starting with the outer end, in the direction of the disposable applicator.

When the reservoir has been fully squeezed, the yellow blister area will be markedly convex toward the outside. This clearly visible convexity indicates that the blister has been properly activated.

- Squeeze the red reservoir until it is completely empty and carefully fold it at the yellow reservoir to the yellow reservoir.

→ To prevent the fluid from flowing back, keep the red reservoir squeezed together while folding.

- The red and yellow blister sections are now on top of each other and the green area of the blister is visible.

- Beginning at the outer end again, squeeze all of the yellow section into the green section of the blister. Use measured pressure and cover the blister opening at the shaft of the applicator with your thumb. This will prevent the contents from spraying out and coming into contact with skin and clothing.

As soon as the fluid has flowed into the green area of the blister, turn the disposable applicator back and forth in the fluid. This will mix the fluid again and cover the applicator back completely.

- When the application of the applicator can be folded while it is still in the blister. In order to do this, only pull the disposable applicator out of the blister far enough to that the intended folding area (narrow section of the applicator shaft) can be seen at the edge of the fold, and fold the applicator at this location over your thumb nail.

- When the fluid is optimally mixed, it will have a homogeneous, yellowish color without streaks. Treat each tooth individually and quickly, as described under Application.

During treatment, move the fluid into the open end by fully squeezing out the green blister area. This allows the applicator to be recast without covering the applicator shaft with fluid. However, if the shaft does become covered with fluid, be sure that the fluid-covered applicator shaft does not come into contact with the mucosa.

### Application

**Instructions for direct light-cure restorations in enamel and dentin:**

- Brush the adhesive onto the surface to be sealed and massage it in for 20 seconds, **applying pressure; longer massaging times are required for larger surfaces.**

- This time cannot be shortened. It is not enough just to brush it on and distribute it.
- The adhesive must be massaged in over the entire surface, as this is the step in which the cavity surface is also etched, conditioned, and covered with adhesive. The more thoroughly the adhesive is massaged in, the more effective it will be.
- Protect the disposable applicator from light during treatment.
- If the applicator is contaminated with blood or saliva, use a new Adper Prompt L-Pop or a new disposable applicator.
- Use a gentle stream of air to thoroughly dry the adhesive to a thin film.
- Apply a second coat of adhesive to a thin film.
- Repeat the brush tip with adhesive and apply a second coat. This coat does not need to be massaged into the surface.
- Agan, use a gentle stream of air to thoroughly dry the adhesive to a thin film.
- Repeat the brush tip with adhesive and apply a second coat. This coat does not need to be massaged into the surface.
- Agan, use a adhesive stream of air to thoroughly dry the adhesive to a thin film.
- Repeat the brush tip with adhesive and apply a second coat. This coat does not need to be massaged into the surface.
- Agan, use a gentle stream of air to thoroughly dry the adhesive to a thin film.
- Repeat the brush tip with adhesive and apply a second coat. This coat does not need to be massaged into the surface.
- Agan, use a adhesive stream of air to thoroughly dry the adhesive to a thin film.
- Repeat the brush tip with adhesive and apply a second coat. This coat does not need to be massaged into the surface.

**Instructions for bonding light-cured dental sealants:**

- Brush the adhesive onto the surface to be sealed and massage it in for 20 seconds, **applying pressure; longer massaging times are required for larger surfaces.**

→ This time cannot be shortened. It is not enough just to brush it on and distribute it.

The adhesive must be massaged in over the entire surface, as this is the step in which the cavity surface is also etched, conditioned, and covered with adhesive. The more thoroughly the adhesive is massaged in, the more effective it will be.

- Protect the disposable applicator from light during treatment.
- If the applicator is contaminated with blood or saliva, use a new Adper Prompt L-Pop or a new disposable applicator.
- Use a gentle stream of air to thoroughly dry the adhesive to a thin film.
- Apply a second layer of adhesive to a thin film.
- Repeat a light-cured sealant.
- Repeat the brush tip with adhesive and apply a second coat. This coat does not need to be massaged into the surface.
- Agan, use a adhesive stream of air to thoroughly dry the adhesive to a thin film.
- Repeat the brush tip with adhesive and apply a second coat. This coat does not need to be massaged into the surface.
- Agan, use a gentle stream of air to thoroughly dry the adhesive to a thin film.
- Repeat the brush tip with adhesive and apply a second coat. This coat does not need to be massaged into the surface.
- Agan, use a adhesive stream of air to thoroughly dry the adhesive to a thin film.
- Repeat the brush tip with adhesive and apply a second coat. This coat does not need to be massaged into the surface.
- Agan, use a gentle stream of air to thoroughly dry the adhesive to a thin film.
- Repeat the brush tip with adhesive and apply a second coat. This coat does not need to be massaged into the surface.
- Agan, use a adhesive stream of air to thoroughly dry the adhesive to a thin film.
- Repeat the brush tip with adhesive and apply a second coat. This coat does not need to be massaged into the surface.

**Notes** It is extremely important for adhesion that both adhesive components be completely mixed. Using incompletely activated Adper Prompt L-Pop blisters can cause the adhesive effect to be eliminated. Massaging the adhesive only partially or shortening the massage time also compromises adhesion.

**Disposal** Only dispose of completely empty packages. The residue of the etching fluids – if heavily diluted with water or neutralized – can be poured down the drain.

**Incompatibilities** In sensitive individuals, sensitization to the product cannot be excluded. If allergic reactions occur, use of the product should be discontinued and the product completely removed. In rare cases, particularly when the instructions for use are not followed, pulp discoloration is possible.

**Storage and Shelf Life** Adper Prompt L-Pop blisters should be protected from light. Do not store the product at temperatures above 25°C. Do not use after the expiration date.

**Customer Information** No person is authorized to provide any information which deviates from the information provided in this instruction sheet.

**Warranty** 3M ESPE warrants this product will be free from defects in material and manufacture. 3M ESPE MAKES NO OTHER WARRANTIES INCLUDING ANY IMPLIED WARRANTY OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE. User is responsible for determining the suitability of the product for intended application. If this product is defective within the warranty period, your exclusive remedy and 3M ESPE’s sole obligation shall be repair or replacement of the 3M ESPE product.

**Limitation of Liability** 3M ESPE will not be liable for any loss of profit or consequential damages, whether direct, indirect, special, incidental or consequential, regardless of the theory asserted, including warranty.

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