3M ESPE	
RelyX™ Temp NE	
Temporary Cement – Zinc Oxide Non-Eugenol	
Цинкоксидний безевгенольный цемент для временной фиксации	
Цимент за временно циментиране без евгенол	
Cink-oksíd cement bez eugenola za privremeno cementiranje protetskih nadomjestaka	
Eugenolot nem tartalmazó ideiglenes cinkoxid cement	
Tymczasowy cement na bazie tlenku cynku bez zawartości eugenolu	
Ciment temporar pe bază de oxid de zinc, fără eugenol	
Dočasný cement na báze oxidu zinočnatého bez eugenolu	
Zink oksidov začasni cement brez evgenola na osnovi cinkovega oksida	
Provizorní cement bez eugenolu	
Geçici Siman-Çinko Oksit-Öjenolsüz	
Eugenooli mittesisaldav ajutine tsinkoksiidsement	
Pagaidu cements – cinka oksīds, nesatur eigenolu	
Laikinas cinko oksido cementas be eugenolio	

Instructions for Use

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Upute za upotrebu

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Instruçiuni de Utilizare

Made in Germany by 3M ESPE AG	CE	0123
Dental Products		
D-82229 Seefeld – Germany		
3M ESPE		
Dental Products		
St. Paul, MN 55144-1000		

3M ESPE Technical Hotline/MSDS Information in U.S.A. and Canada 1-800-634-2249.

3M, ESPE and RelyX are trademarks of 3M or 3M ESPE AG. © 3M 2003. All rights reserved.

ENGLISH

Description of Product

RelyX™ Temp NE, manufactured by 3M ESPE, is an eugenol-free self-curing cement to be used for temporary fixation of dental restorations. By adding Vaseline the final hardness of the cement can be reduced for easier removal of bridges, for example. RelyX Temp NE **does not** affect the polymerization of filling and luting composites. RelyX Temp NE is especially suitable for use on patients with sensitivity to eugenol.

The information for use of this product has to be kept for the duration of the application.

Fields of Application

Temporary fixation of provisional restorations.

Temporary fixation of crowns, bridges, inlays, and onlays.

Preparation

Use a water spray to clean the cavity surfaces and dry them prior to cementation. Water and alcohol remnants in the cavity will accelerate the setting process.

Protection of the Pulp

In case of deep cavities apply the protection of the pulp usually employed for temporary fixation.

Dispensing and Mixing

Use RelyX Temp NE at an ambient temperature of 18° to 23°C/64° to 73°F and a relative humidity of 45 to 55%. Dispense two equal lengths of base paste and catalyst on a mixing pad. Use a metal or plastic spatula to thoroughly mix the pastes for 30 seconds until a homogeneous consistency is achieved. By overdosing the catalyst, the bonding strength is increased and the working time is prolonged.

Close the tubes immediately after use. Do not mix up the tube tops. For temporary fixation of final restorations, the final hardness of the cement has to be reduced by adding Vaseline, at the same time the working time is extended. If a length of 0.5 cm of base paste (86 mg) and of catalyst (39 mg) each is used, a maximum of 42 mg (lentil size) of Vaseline can be admixed.

Application

Apply a thin layer of cement on the surface to be cemented, and place the dental restoration with slight pressure.

Times

The following times apply for an ambient temperature of 23°C/73°F:
with maximum amount of Vaseline

Mixing	min. : sec.	min. : sec.
Working after start of mixing	0 : 30	0 : 30
Setting after start of mixing	1 : 30	2 : 30
	3 : 30	4 : 00

Setting is accelerated over an ambient temperature of 23°C/73°F and a relative humidity of over 55%.

Removal of Excessive Material

Use an instrument to remove any excessive material after setting has occurred.

Note

In order to prevent setting of the paste in the tube due to paste carry-over, do not mix up the screw tops.

Incompatibilities

In susceptible individuals, sensitization to the product cannot be excluded. Use of the product should be discontinued and the product completely withdrawn from contact with the patient, if allergic reactions are observed. There may be rare cases of pulpal sensitivity, especially if the Instructions for Use were not observed.

Storage and Shelf-life

Do not store the product above 25°C/77°F. Do not use after the expiry date.

Customer Information

No person is authorized to provide any information which deviates from the information provided in this instruction sheet.

Warranty

3M ESPE warrants this product will be free from defects in material and manufacture. 3M ESPE MAKES NO OTHER WARRANTIES INCLUDING ANY IMPLIED WARRANTY OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE. User is responsible for determining the suitability of the product for user’s application. If this product is defective within the warranty period, your exclusive remedy and 3M ESPE’s sole obligation shall be repair or replacement of the 3M ESPE product.

Limitation of Liability

Except where prohibited by law, 3M ESPE will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the theory asserted, including warranty, contract, negligence or strict liability.

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